



 <p>“YELLOW TOP” BD UA No Additive 8mL</p>	<p>Urinalysis, Dipstick Only Urinalysis with Microscopic Urinalysis with Microscopic & Culture if indicated Urine Eosinophils HCG, Urine</p>	<p>FRIDGE 24HRS 2mL minimum</p>
 <p>“TIGER TOP” BD UA Preservative Tube 8mL</p>	<p>Urinalysis, Dipstick Only Urinalysis with Microscopic Urinalysis with Microscopic & Culture if indicated Urine Eosinophils</p>	<p>ROOM TEMP 72HRS Note minimum/maximum fill lines</p>
 <p>“YELLOW PRESERVE” BD UA Preservative Tube 8mL</p>	<p>Urinalysis, Dipstick Only Urinalysis with Microscopic Urinalysis with Microscopic & Culture if indicated Urine Eosinophils</p>	<p>ROOM TEMP 72HRS Note minimum/maximum fill lines</p>
 <p>“GRAY TOP” BD Urine C&S Preservative (Boric Acid) 4mL</p>	<p>Culture, Urine Urinalysis with Microscopic & Culture if indicated If QNS to meet minimum fill, use a red top (below) for culture</p>	<p>ROOM TEMP 72HRS 3mL minimum fill MUST be met or will be rejected, sterile transfer</p>
 <p>“RED TOP” BD No Additive 3mL or 6mL</p>	<p>HCG OSMO LEGIONELLA S. PNEUMO NICOTINE DRUG SCREEN CREATININE SODIUM (NA)</p> <p>POTASSIUM (K) CHLORIDE (CL) CALCIUM (CA) URIC ACID AMYLASE TOTAL PROTEIN (TP) MICROALBUMIN (uALB) UREA NITROGEN (UUN)</p>	<p>REFRIGERATE 0.5mL minimum will get this entire battery of tests done</p>
	<p>CULTURE only acceptable for culture if not contaminated by other testing</p>	<p>FRIDGE 24HRS 5 drops minimum, sterile transfer</p>

Please refer to test directory for further information and tests not listed, or call the main lab at 625-5800

URINALYSIS WITH MICROSCOPIC
URINALYSIS WITH MICROSCOPIC & CULTURE IF INDICATED =

